Starta TD Control



Klicka på **alternativ**



Klicka på inställningar



Gå till sidan2 genom att klicka på den pilen längst till höger

्			
Aktivering	Kalibrering	Tastatur	
Konfigurer sporingssirkelen og knappene dine. Inkluderer fokuseringstid, aktiveringstype, sporingssirkelens synlighet og mer.	Kalibrering og konfigurering Inkluderer kalibrering, styringsstatus og mer.	Konfigurer tastaturet og ordprediksjonene dine. Inkluderer fokuseringstid, språk, tilbakerneldingslyd og mer.	>
Apne	Apne	Apne	

Klicka på öppna Modus

8	¢¢	
Modus	Generelt	Tilpassing
Endre programmodusen til å passe aktiviteten din.	Generell informasjon om Control. Inkluderer programspråk, versjonsnummer og mer.	Tilpass opplevelsen med Control.
Apne	Apne	Apne

Ställ in enligt nedan







Nu ska det se ut som nedan



När man har gjort dessa inställningar så koppla in ett testatur och klicka på **F2 på tastaturet,** da skjules menyen (inställningarna blir kvar och brukaren kan använda program utan att störas av menyn där nere)